

How to Assign a Biron Pharmacogenomic Test



Obtaining and using a pharmacogenomic test is straightforward. It will help you choose effective medications and reduce the risk of adverse side effects.

Option 1 : Kit shipped at home



1

Requisition

Fill in the requisition and give it to the patient.



2

Kit order

Order the test online at biron.com/pgx.
The patient can perform this step themselves or be accompanied by the clinic staff.



3

At home delivery

The sampling kit is shipped directly to the patient's home.



4

Sampling and return

The patient proceeds to the saliva sampling by following the instructions included in the kit. They return the sample, the requisition, and the consent (if applicable) using the provided envelop and dropping it off in a regular mailbox.



5

Consulting results

An email notification is sent as soon as the results are ready (about 5 business days after sample received at lab). Log on to the Biron | BiogeniQ portal to consult the report. A copy can be printed directly from the portal.

For any questions regarding interpretation, contact Michel Cameron at mcameron@biron.com or at 514-616-8850.



6

Patient access to results

At follow-up, inform patient they can access their report by login to their Biron | BiogeniQ portal.

If you have any questions or concerns, please contact us at pgxinfo@biron.com or 1-866-923-9222, ext. 8723. We are here to help!

How to Assign a Biron Pharmacogenomic Test



Obtaining and using a pharmacogenomic test is straightforward. It will help you choose effective medications and reduce the risk of adverse side effects.

Option 2 : Inventory at the clinic



1

Requisition

Fill in the requisition and give it to the patient



2

Kit assignment

Go online to assign a kit to patient by following instructions in the box.



3

Payment

Click on **Online payment** by credit card to complete the kit assignment. Enter the patient payment information in the secured «**Stripe**» window that appears.



4

Sampling and return

Proceed to the saliva sampling by following the instructions included in the kit. Return the sample, the requisition, and the consent (if applicable) using the provided envelop. Drop it off in a regular mailbox.



5

Consulting the results

An email notification is sent as soon as the results are ready (about 5 business days after sample received at lab). Log on to the Biron | BiogeniQ portal to consult the report. A copy can be printed directly from the portal.

For any questions regarding interpretation, contact Michel Cameron at mcameron@biron.com or at 514-616-8850.



6

Patient access to results

At follow-up, inform patient they can access their report by login to their Biron | BiogeniQ portal.