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02

03

powered by  
Intelrad®  
Distributed Radiology Solutions™

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The screenshot shows the InteleConnect login interface. At the top left is the InteleConnect logo (a red 3D cube) and the text 'InteleConnect by Intelerad'. Below this, the text 'InteleConnect PACS-4-11-1-R25' is visible. The main content area includes the URL 'rx.imagixmedical.com', a language selector with 'English' selected, and a link 'Demander un nouveau compte'. The login fields are labeled 'Nom d'utilisateur' and 'Mot de passe', with a blue arrow button to the right. A grey error message box at the bottom left contains the text: 'Votre nom d'utilisateur ou mot de passe est incorrect. Veuillez essayer de nouveau. Avez-vous oublié votre mot de passe?'. The Intelerad logo is in the bottom right corner.

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2/3

Access Break Glass

This feature allows you to view studies for which you were not granted automatic access.

I understand that I am about to access information for a patient for whom I am not the primary referring physician on record. I acknowledge that I am doing so only for medical reasons and for the sole benefit of the patient, and in doing so am bound to keep this information private and confidential.

Note: This feature allows you to view only studies currently available on the PACS. You do not have access to studies for this patient that are acquired at a later date.

Please allow a few minutes for the studies to become available.

I acknowledge and accept the terms of this confidentiality agreement.

Cancel

**04**

I acknowledge and accept the terms of this confidentiality agreement.

Provide the full patient ID

Or

Provide the full patient name and the date of birth

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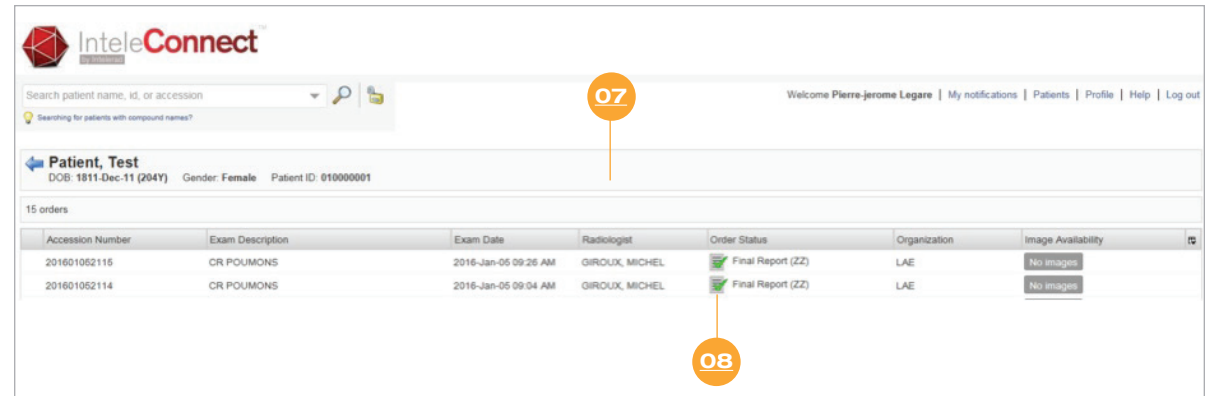
3/3

The screenshot displays the IntelConnect web interface. At the top, there is a search bar with the text "Search patient name, id, or accession" and a magnifying glass icon. To the right of the search bar, there is a welcome message: "Welcome Pierre-Jerome Legare | My notifications | Patients | Profile | Help | Log out". Below the search bar, there is a section for "Patient, Test" with the following details: "DOB: 1811-Dec-11 (204Y) Gender: Female Patient ID: 010000001". Below this, there is a table with the heading "15 orders". The table has the following columns: "Accession Number", "Exam Description", "Exam Date", "Radiologist", "Order Status", "Organization", and "Image Availability". The table contains two rows of data:

Accession Number	Exam Description	Exam Date	Radiologist	Order Status	Organization	Image Availability
201601052115	CR POUMONS	2016-Jan-05 09:26 AM	GIROUX, MICHEL	Final Report (ZZ)	LAE	No images
201601052114	CR POUMONS	2016-Jan-05 09:04 AM	GIROUX, MICHEL	Final Report (ZZ)	LAE	No images

Two callout boxes are present: "07" is located above the search bar, and "08" is located below the "Image Availability" column of the table.

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The screenshot displays the IntelConnect web interface. At the top, the IntelConnect logo is visible. Below it is a search bar with the text "Search patient name, id, or accession" and a search icon. To the right of the search bar, there are navigation links: "Welcome Referring Physician", "My notifications", "Patients", "Profile", "Help", and "Log out".

The main content area shows patient information for "Brown, Chris Gordon" with details: "DCB: 1972-Mar-12 (43Y)", "Gender: Male", and "Patient ID: 10037". There are several action buttons: "View PDF...", "Launch IntelViewer", "Grant Access...", "Add Impression...", and "Edit". A red circle with the number "09" is positioned over the "Launch IntelViewer" button.

Below the patient information, there is a section titled "CR CHEST 2 VIEWS FRONTAL AND LATERAL". It lists the following details:

- Priority: Routine
- Accession Number: 10160
- Referring Physician: Referring Physician
- Organization: Entering Organization
- Status: Final Report (ZZ)
- Study Date: 2004-Oct-12 03:15 PM
- Radiologist: Resident, John (with a "Start a Chat" link)
- Location: AssignedPatient.Location

Below this section is a "Final Report (ZZ)" section with the following text:

CHEST 2 VIEWS FRONTAL AND LATERAL: 1522 HOURS  
INDICATION: Cough.  
FINDINGS: There is an old healed rib fracture on the left. No infiltrates or nodules are seen. The heart size is normal. The hila, mediastinum and pleura are unremarkable.  
IMPRESSION: Negative chest exam.  
Dr. XXXXX 10/12/2004 AddendumRoger

On the right side of the interface, there is a warning message: "Images in IntelConnect are intended for review only and are not appropriate for diagnostic purposes. Please use IntelViewer for diagnostic viewing." Below this message are two chest X-ray images: "CR PA (1 image)" and "CR LAT (1 image)". A red circle with the number "09" is also positioned over the "CR PA" image.

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
The image displays two screenshots of the IntelConnect web application interface. The top screenshot shows the main navigation bar with the IntelConnect logo and a search bar. A callout '01' points to the 'Profile' link in the top right corner. The bottom screenshot shows the 'Profile' page with a 'Save' button and a 'Profile' tab. A callout '02' points to the 'Account' sub-tab. A callout '03' points to the 'Required' section of the profile form, which includes fields for Username, First name, Last name, Email address, Secondary email address, Primary telephone, Fax number, Organization, Account Requested by, and Account Created by. Below this is the 'Session Timeout' section with a checkbox and a dropdown menu, and the 'Password' section with a 'Change password' checkbox and input fields for 'Current password' and 'New password'.

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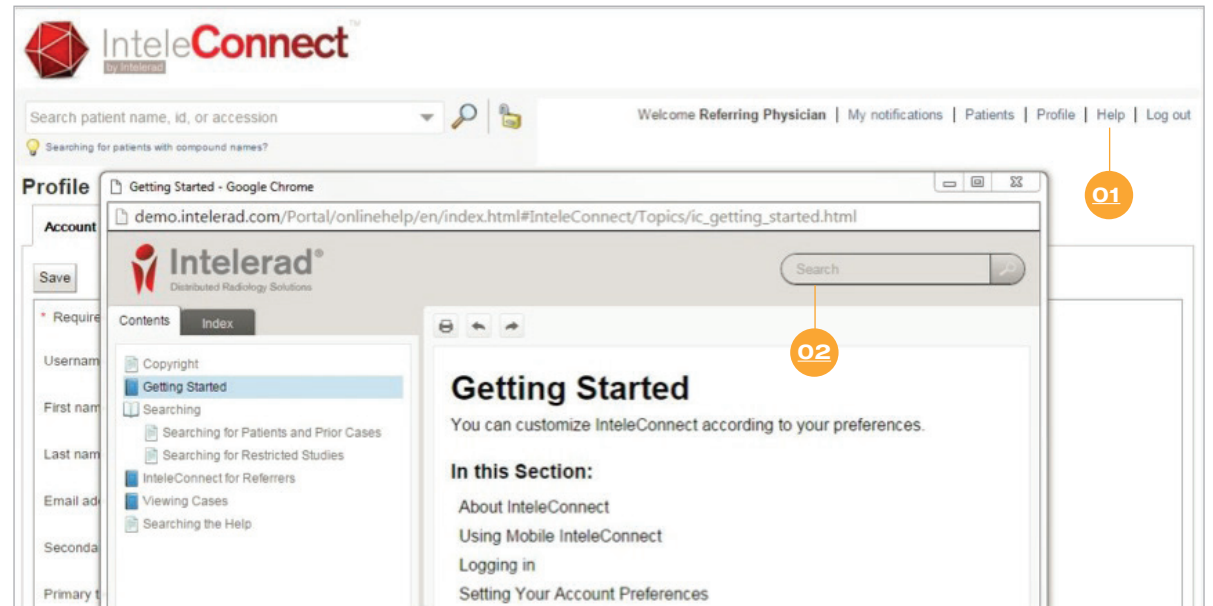The screenshot shows the IntelConnect web interface. At the top, there is a search bar with the text "Search patient name, xl, or accession" and a "Searching for patients with compound names?" link. The user is logged in as "Welcome Referring Physician" with links for "My notifications", "Patients", "Profile", "Help", and "Log out". Below the search bar, there is a "My notifications (15 new)" section. The main content area is titled "Edit List Parameters" and contains several filter options:

- List name:** A text input field containing "Patients by location".
- Show in worklist toolbar:** A star icon and a checkbox.
- Date range:** A dropdown menu set to "Last 7 days".
- Exam Status:** An empty text input field.
- Modality:** An empty text input field.
- Patients:** Radio buttons for "All" (selected) and "Only those I referred".
- Discrepancies & critical results:** Checkboxes for "Show only patient orders with"  "Impression discrepancies" and  "Critical results".
- Sorting:** A dropdown menu set to "Most recent".
- Organizations:** An empty text input field.
- Locations:** An empty text input field.

At the bottom of the dialog, there are two buttons: "Save and Apply" and "Cancel".

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