

# Philips Respironics Dreamstation 2



! We strongly recommend that you clean the mask before first use.

## Every day



Change the water in your **water chamber**.  
Use only distilled or demineralized water.<sup>1</sup>



Wash the **pillow or cushion** of your mask in warm soapy water or with a wipe intended for this purpose.

## Every week



Wash in warm soapy water.  
Mask, water chamber, breathing tube



Clean the **hypoallergenic filter** under water (no soap).

## Every month



Change the **filter**.  
Ultra-fine filter: Monthly

## Every six months



If you don't change your **mask twice a year**, replace your **cushion every 6 months**.



Change the **tubing and the filter**.  
Hypoallergenic filter: Every 6 months

## Every year



Change the **water chamber** every year.



Replace the **mask** every year, ideally every six months.



Replace any equipment that is cracked, tarnished or damaged.

Even with normal use and maintenance, it is impossible to prevent the growth of certain microorganisms. For this reason, we recommend that you replace the accessories **regularly** to ensure that your treatment is effective.

1. Available at all drugstores and at certain grocery stores and big-box stores.