

Philips Respironics Dreamstation



! We strongly recommend that you clean the mask before first use.

Every day



Change the water in your water chamber.

Use only distilled or demineralized water ¹.



Wash the pillow or cushion of your mask in warm soapy water or with a wipe intended for this purpose.

Every week



Wash in warm soapy water

Mask, water chamber, breathing tube



Clean the hypoallergenic filter under water (no soap)

Every month



Change the filter

Ultrafine: Every month

Every six months



If you don't change your mask twice a year, replace your cushion every 6 months



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
Change the filter

Hypoallergenic: Every 6 months

Every year



Change the breathing tube every year and the water chamber when required, unless it is disposable.



Replace the mask every year, ideally every six months.



Replace any equipment that is cracked, tarnished or damaged.

Even with normal use and maintenance, it is impossible to prevent the growth of certain microorganisms. For this reason, we recommend that you replace the accessories regularly to ensure that your treatment is effective.

¹. Available at all drugstores and at certain grocery stores and big-box stores.